



Cosentyx[®]
(secukinumab)



UnoReady[®] Pen
(300 mg)

Sensoready[®] Pen
(150 mg)

QUICK TIPS FOR USING YOUR AUTO-INJECTOR UNOREADY PEN OR SENSOREADY PEN

INDICATIONS

COSENTYX[®] (secukinumab) is a prescription medicine used to treat:

- people 6 years of age and older with moderate to severe plaque psoriasis (PsO) that involves large areas or many areas of the body, and who may benefit from taking injections or pills (systemic therapy) or phototherapy (treatment using ultraviolet or UV light alone or with systemic therapy)
- people 2 years of age and older with active psoriatic arthritis (PsA)
- adults with active ankylosing spondylitis (AS)
- adults with active non-radiographic axial spondyloarthritis (nr-axSpA) and objective signs of inflammation
- people 4 years of age and older with active enthesitis-related arthritis (ERA)
- adults with moderate to severe hidradenitis suppurativa (HS)

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit [COSENTYX.com](https://www.cosentyx.com) for more information.



"I just get it. It's easy. I don't see the needle."

CYNDI LAUPER

ACTUAL PATIENT WITH
PLAQUE PSORIASIS USING
THE SENSOREADY[®] PEN

This brochure contains photos of actual patients who have been compensated for their time. Individual results may vary.

IMPORTANT SAFETY INFORMATION (cont)

Do not use COSENTYX[®] if you have had a severe allergic reaction to secukinumab or any of the other ingredients in COSENTYX. See the Medication Guide for a complete list of ingredients.

What is the most important information I should know about COSENTYX?

COSENTYX is a medicine that affects your immune system. COSENTYX may increase your risk of having serious side effects such as:


Infections

COSENTYX may lower the ability of your immune system to fight infections and may increase your risk of infections. Some people have had serious infections while taking COSENTYX, including tuberculosis (TB), and infections caused by bacteria, fungi, or viruses. Some people have died from these infections.


- Your doctor should check you for TB before starting treatment with COSENTYX.
- If your doctor feels that you are at risk for TB, you may be treated with medicine for TB before you begin treatment with COSENTYX and during treatment with COSENTYX.
- Your doctor should watch you closely for signs and symptoms of TB during treatment with COSENTYX. **Do not use COSENTYX if you have an active TB infection.**

GETTING COMFORTABLE WITH YOUR NEW PEN

This brochure has the Instructions for Use for 2 different pens, so make sure you identify which pen you have been prescribed—the COSENTYX UnoReady[®] 300 mg/2 mL Pen or the COSENTYX Sensoready[®] 150 mg/1 mL Pen.

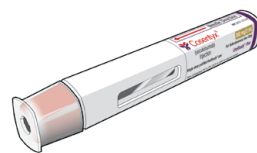

UnoReady Pen

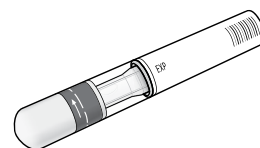

Sensoready Pen

Now that your doctor has shown you how to use your pens, here are a few things you should keep in mind.

FILL THIS OUT WITH YOUR DOCTOR SO YOU'RE CLEAR ABOUT YOUR DOSE

MY DOSE IS:

150 mg

taken with 1 Sensoready Pen

300 mg

taken with 1 UnoReady Pen

300 mg

taken with 2 Sensoready Pen injections

I TAKE COSENTYX:

1x a week for the first 5 weeks and then 1x every 4 weeks after that

1x every 4 weeks

1x every 2 weeks (if applicable)

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

IMPORTANT REMINDERS

Take your injection pen out of the refrigerator ahead of time, so that it's at room temperature. Setting a timer can help.



The **UnoReady[®] Pen** needs to be taken out of the fridge **30–45 minutes** before your injection.

The **Sensoready[®] Pen** needs to be taken out of the fridge **15–30 minutes** before your injection.

Clean the skin where you plan to inject with an alcohol wipe before using the injection pen.



The cap to the **UnoReady Pen pulls off**. For the UnoReady Pen Instructions for Use, see page 10.

The cap to the **Sensoready Pen twists off**. For the Sensoready Pen Instructions for Use, see page 22.

When injecting with your injection pen, remember to **“Click, Watch, Click & Wait.”**

STEP 1 CLICK

Use the COSENTYX[®] pen within 5 minutes of removing the cap. Hold the pen at 90 degrees against the cleaned injection site, then press and hold it firmly against your skin. The first click indicates the injection has started.

STEP 2 WATCH

A green indicator will start moving within the viewing window of your injection pen. It's important to keep holding the pen firmly against your skin while you watch the green indicator and listen for the second click.

STEP 3 CLICK & WAIT

The second click indicates the injection is almost complete. After you hear the second click, **wait for 5 more seconds** before removing the pen. Make sure the green indicator has completely filled the window and stopped moving. You can then remove the injection pen.



“I have the same routine once a month. Take my COSENTYX at the kitchen table, throw out my pen in my sharps container, and boom, I'm good to go.”

GARY

ACTUAL PATIENT WITH PLAQUE PSORIASIS AND PSORIATIC ARTHRITIS USING THE SENSOREADY PEN

IMPORTANT SAFETY INFORMATION (cont)

Before starting COSENTYX, tell your doctor if you:

- are being treated for an infection
- have an infection that does not go away or that keeps coming back
- have TB or have been in close contact with someone with TB
- think you have an infection or have symptoms of an infection such as: fevers, sweats, or chills; muscle aches; cough; shortness of breath; blood in your phlegm; weight loss; warm, red, or painful skin or sores on your body; diarrhea or stomach pain; burning when you urinate or urinate more often than normal

After starting COSENTYX, call your doctor right away if you have any signs of infection listed on pages 2 or 5.

Do not use COSENTYX if you have any signs of infection unless you are instructed to by your doctor.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit [COSENTYX.com](https://www.cosentyx.com) for more information.

COSENTYX® CONNECT IS HERE FOR YOU THROUGHOUT YOUR TREATMENT.

COSENTYX® Connect is a personalized support program for people taking or considering COSENTYX®. Sign up now for access to a full range of personalized services, like a **COSENTYX® Connect Team Member**, a co-pay of as little as \$0* if privately insured and otherwise eligible, and **injection resources** to help make your experience with COSENTYX as easy, affordable, and convenient as possible.

Whether it's:

- getting started with a specialty pharmacy
- help understanding your insurance coverage
- savings options and more

A COSENTYX® Connect Team Member is here to help guide you throughout your treatment with COSENTYX.

*Limitations apply. Valid only for those with private insurance. Program provides up to \$16,000 annually for the cost of COSENTYX and up to \$150 per infusion (up to \$1,950 annually) for the cost of administration. Co-pay support for infusion administration cost not available in Rhode Island or Massachusetts. Offer not valid under Medicare, Medicaid, or any other federal or state program. Novartis reserves the right to rescind, revoke, or amend this program without notice. See complete Terms & Conditions for details.

Sign up now at [COSENTYX.com/signup](https://www.cosentyx.com/signup) or call us at **1-844-COSENTYX**



“Having COSENTYX® Connect walk me through everything that first year that I was taking the medication really helped me feel more at ease and made things very simple.”

WENDY

ACTUAL PATIENT WITH ANKYLOSING SPONDYLITIS USING THE SENSOREADY® PEN

SUPPORT

UNREADY PEN INSTRUCTIONS

SENSOREADY PEN INSTRUCTIONS

IMPORTANT SAFETY INFORMATION (cont)

What are the possible side effects of COSENTYX?

COSENTYX may cause serious side effects, including:

Serious allergic reactions

Serious allergic reactions can occur. Get emergency medical help right away if you get any of the following symptoms: feeling faint; swelling of your face, eyelids, lips, mouth, tongue, or throat; trouble breathing or throat tightness; chest tightness; skin rash or hives (red, itchy bumps).

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit [COSENTYX.com](https://www.cosentyx.com) for more information.

THINGS TO CONSIDER WHEN USING YOUR UNOREADY PEN

- The COSENTYX[®] UnoReady Pen has a 100% satisfaction rating and delivers a **300-mg dose in a single injection** that is taken at home.*
- In a clinical study, the majority of adults reported **no pain or reaction** with the injection.†
- The UnoReady Pen is convenient, with a streamlined design and a hidden needle.

USE COSENTYX EXACTLY AS PRESCRIBED BY YOUR HEALTHCARE PROVIDER.

If your healthcare provider decides that you or your caregiver may give your injections of COSENTYX at home, you should receive training on the right way to prepare and inject COSENTYX. Do not try to inject until you or your caregiver has been shown how by your healthcare provider. Be sure that you read, understand, and follow the Instructions for Use and Medication Guide, and contact your healthcare provider with any further questions.

*Based on a study in which all 37 participants using the UnoReady Pen reported being satisfied or very satisfied with the device at 28 weeks.

†Based on a study in which 30 of 38 adults with plaque psoriasis using the UnoReady Pen reported no pain with the injection at 12 weeks.

COSENTYX[®] CONNECT IS HERE TO HELP

For questions about using the UnoReady Pen, call **1-844-267-3689** Monday–Friday, 8 AM to 8 PM ET.



WATCH A DEMONSTRATION

Follow along with our UnoReady Pen injection training video at [COSENTYX.com/all/how-to-inject-cosentyx/unoready-pen](https://www.cosentyx.com/all/how-to-inject-cosentyx/unoready-pen)

IMPORTANT SAFETY INFORMATION (cont)

If you have a severe allergic reaction, do not give another injection of COSENTYX.

Inflammatory bowel disease

New cases of inflammatory bowel disease or “flare-ups” can happen with COSENTYX, and can sometimes be serious. If you have inflammatory bowel disease (ulcerative colitis or Crohn’s disease), tell your doctor if you have worsening disease symptoms during treatment with COSENTYX or develop new symptoms of stomach pain or diarrhea.

Severe skin reactions that look like eczema can happen during treatment with COSENTYX from days to months after your first dose and can sometimes lead to hospitalization. Your doctor may temporarily stop treatment with COSENTYX if you develop severe skin reactions. Tell your doctor if you have any of the following signs or symptoms: redness or rash; itching; small bumps or patches; your skin is dry or feels like leather; blisters on the hands or feet that ooze or become crusty or skin peeling.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit [COSENTYX.com](https://www.cosentyx.com) for more information.

THE UNOREADY PEN: INSTRUCTIONS FOR USE

COSENTYX[®] (koe-sen'-tix) (secukinumab) Injection 300 mg/2 mL UNOREADY PEN

Be sure that you read, understand, and follow this Instructions for Use before injecting COSENTYX. Your healthcare provider should show you how to prepare and properly inject with the COSENTYX single-dose UnoReady Pen before you use it for the first time. Talk to your healthcare provider if you have any questions.

IMPORTANT:

- **Do not use** the COSENTYX UnoReady Pen if the seal on the outer carton is broken. Keep the COSENTYX UnoReady Pen in the sealed outer carton until you are ready to use it.
- **Do not shake** the COSENTYX UnoReady Pen.
- If you drop your COSENTYX UnoReady Pen, **do not use** it if the pen looks damaged, or if you dropped it with the cap removed.
- The needle is covered by the needle guard and the needle will not be seen. **Do not** touch or push the needle guard because you could get a needle stick.
- Throw away (dispose of) the used COSENTYX UnoReady Pen right away after use.
- **Do not re-use the COSENTYX UnoReady Pen.** See “**Step 9. Disposing of the used COSENTYX UnoReady Pen**” at the end of this Instructions for Use.

HOW SHOULD I STORE THE COSENTYX 300 mg/2 mL UNOREADY PEN?

- Store your carton of COSENTYX UnoReady Pens in a refrigerator, between 36°F to 46°F (2°C to 8°C).
- Keep the COSENTYX UnoReady Pen in the original carton until ready to use to protect from light.
- **Do not** freeze the COSENTYX UnoReady Pen.
- Throw away (dispose of) any unused COSENTYX UnoReady Pens.

Keep the COSENTYX UnoReady Pen and all medicines out of the reach of children.

COSENTYX UNOREADY PEN PARTS

(see Figure A):

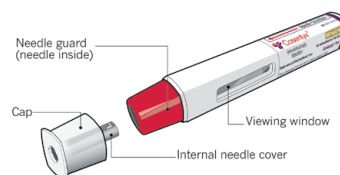


Figure A

The COSENTYX UnoReady Pen is shown above with the cap removed. **Do not** remove the cap until you are ready to inject.

IMPORTANT SAFETY INFORMATION (cont)

The most common side effects of COSENTYX include: cold symptoms, diarrhea, and upper respiratory tract infections.

These are not all of the possible side effects of COSENTYX. Call your doctor for medical advice about side effects.

Before using COSENTYX, tell your doctor if you:

- have any of the conditions or symptoms listed on page 5 for infections.
- have inflammatory bowel disease (Crohn's disease or ulcerative colitis).
- are allergic to latex. The needle cap on the COSENTYX Sensoready[®] pen, and 150 mg/mL and 75 mg/0.5 mL prefilled syringes contains latex.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

WHAT YOU NEED FOR YOUR INJECTION:

Included in the carton (see Figure B):

- A new COSENTYX® UnoReady Pen.
- Each** COSENTYX UnoReady Pen contains **300 mg** of COSENTYX. Check to make sure that you have the correct medicine and dose.

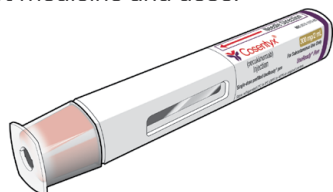


Figure B

Not included in the carton (see Figure C):

- Alcohol wipe
- Cotton ball or gauze
- Sharps disposal container

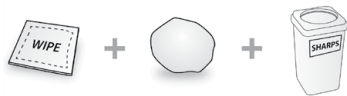


Figure C

See “How should I dispose of used COSENTYX UnoReady Pens?” (STEP 9) at the end of this Instructions for Use.

BEFORE YOUR INJECTION:

STEP 1. Bring to room temperature.

Take the carton containing the COSENTYX UnoReady Pen out of the refrigerator (see Figure D) and leave it **unopened for 30 to 45 minutes before injecting** to allow it to reach room temperature.

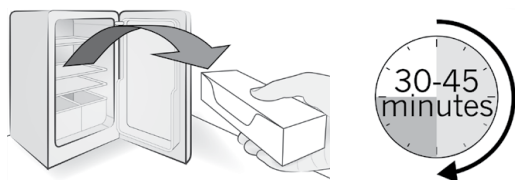


Figure D

STEP 2. Important safety checks before you inject (see Figure E).

- Look at the expiration date (**EXP**) on your COSENTYX UnoReady Pen. **Do not use** the UnoReady Pen if the expiration date has passed.
- Look through the viewing window. The liquid should be clear. Its color may vary from colorless to slightly yellow.
- **Do not** use if the liquid contains visible particles, or is cloudy or discolored. You may see air bubbles, which is normal.

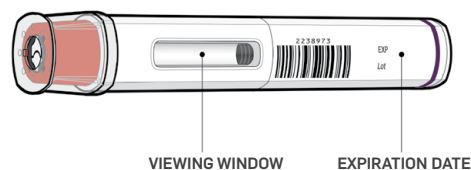


Figure E

Contact your healthcare provider or pharmacist if the COSENTYX UnoReady Pen fails any of these checks.

IMPORTANT SAFETY INFORMATION (cont)

- have recently received or are scheduled to receive an immunization (vaccine). People who take COSENTYX **should not** receive live vaccines. Children should be brought up to date with all vaccines before starting COSENTYX.
- have any other medical conditions and all the medicines you take, including prescription and over-the-counter medicines, vitamins, and herbal supplements. Keep a list of your medicines to show your healthcare provider and pharmacist when you get a new medicine.
- are pregnant or plan to become pregnant. It is not known if COSENTYX can harm your unborn baby. You and your doctor should decide if you will use COSENTYX.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

STEP 3. Choose your injection site:

- The recommended site is the front of the thighs. You may also use the lower abdomen, but **not** the area 2 inches around the navel (belly button) (see Figure F).
- Choose a different site each time you give an injection.
- **Do not** inject into areas where the skin is tender, bruised, red, scaly, or hard or in an area of skin that is affected by psoriasis. Avoid areas with scars or stretch marks.

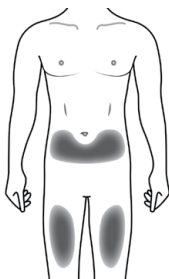


Figure F

- If a **caregiver** or **healthcare provider** is giving you your injection, they may also inject into your outer upper arm (see Figure G).

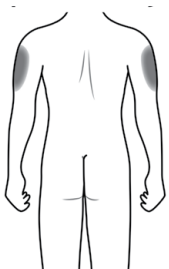


Figure G

STEP 4. Cleaning your injection site:

- Wash your hands well with soap and water.
- Using a circular motion, clean the injection site with the alcohol wipe. Leave it to dry before injecting (see Figure H).
- Do not touch the cleaned area again before injecting.



Figure H

YOUR INJECTION:**STEP 5. Removing the cap:**

- Only remove the cap when you are ready to use the COSENTYX® UnoReady® Pen.
- **Pull** the cap straight off (see Figure I). **Do not** twist the cap.
- Throw away the cap. **Do not try to re-attach the cap** as you may bend the needle.
- Use the COSENTYX UnoReady Pen within 5 minutes of removing the cap.

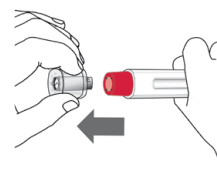


Figure I

Please see Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

STEP 6. Position the COSENTYX® UnoReady Pen:

- Hold the COSENTYX UnoReady Pen at 90 degrees against the cleaned injection site with the viewing window facing you (see Figure J).

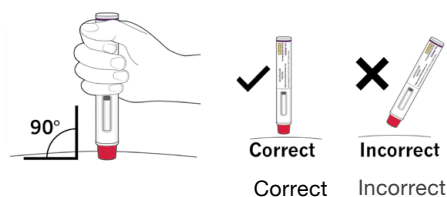


Figure J

STEP 7. Starting your injection:

- Press and hold the COSENTYX UnoReady Pen **firmly** against the skin to start the injection (see Figure K).
- The **1st click** indicates the injection has started.
- **Keep holding** the pen **firmly** against your skin after the 1st click.



Figure K

Monitor the injection:

- The green indicator will move within the viewing window.
- The **2nd click** indicates the injection is **almost** complete.
- **Keep holding** the pen **firmly** against your skin after the 2nd click.



Figure K

- **Keep holding** the pen **firmly** against your skin and slowly count to 5.

After the 2nd click,
continue to hold down
and slowly count to 5

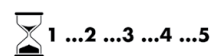


Figure K

Complete the injection.

- Check that the **green indicator with the gray tip** has filled the window and has stopped moving. (see Figure K).
- The COSENTYX UnoReady Pen can now be removed from your skin.

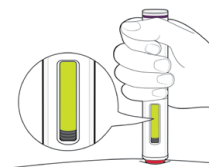


Figure K

Please see Important Safety Information throughout brochure.
Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

AFTER YOUR INJECTION:

STEP 8. Check that the green indicator has filled the window (see Figure L):

- This means the medicine has been delivered. Contact your healthcare provider or pharmacist if the green indicator is not visible or does not fill the window.
- There may be a small amount of blood at the injection site. You can press a cotton ball or gauze over the injection site and hold it for a few seconds. **Do not** rub the injection site. You may cover the injection site with a small adhesive bandage, if needed.

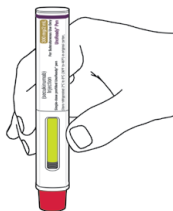


Figure L

How should I dispose of used COSENTYX® UnoReady Pens?

STEP 9: Put your used COSENTYX UnoReady Pen in an FDA-cleared sharps disposal container right away after use (see Figure M). **Do not** throw away (dispose of) the UnoReady Pen in your household trash.

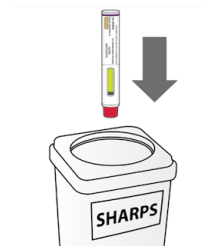


Figure M

If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:

- made of a heavy-duty plastic,
- can be closed with a tight-fitting, puncture-resistant lid without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labeled to warn of hazardous waste inside the container.

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles, syringes, and COSENTYX UnoReady Pens.

For more information about safe sharps disposal, and for specific information about sharps disposal in the state that you live in, go to the FDA's website at: <http://www.fda.gov/safesharpsdisposal>.

Please see Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

THINGS TO CONSIDER WHEN USING YOUR SENSTOREADY PEN

- The COSENTYX[®] Sensoready[®] Pen is designed to be easy to use, with 90% of adults reporting **no pain or reaction** during or after the injection.*
- The cap of the Sensoready Pen contains natural rubber latex.


*In a study of patients with active psoriatic arthritis.

USE COSENTYX EXACTLY AS PRESCRIBED BY YOUR HEALTHCARE PROVIDER.

If your healthcare provider decides that you or your caregiver may give your injections of COSENTYX at home, you should receive training on the right way to prepare and inject COSENTYX. Do not try to inject until you or your caregiver has been shown how by your healthcare provider. Be sure that you read, understand, and follow the Instructions for Use and Medication Guide, and contact your healthcare provider with any further questions.

COSENTYX[®] CONNECT IS HERE TO HELP

For questions about using the Sensoready Pen, call **1-844-267-3689** Monday–Friday, 8 AM to 8 PM ET.

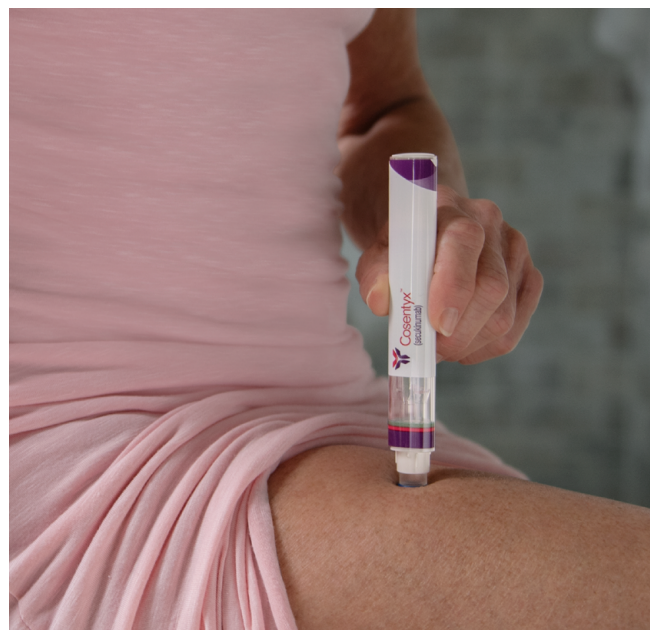

WATCH A DEMONSTRATION

Follow along with our Sensoready Pen injection training video at [COSENTYX.COM/all/how-to-inject-cosentyx/sensoready-pen](https://www.cosentyx.com/all/how-to-inject-cosentyx/sensoready-pen)

IMPORTANT SAFETY INFORMATION (cont)

- are breastfeeding or plan to breastfeed. It is not known if COSENTYX passes into your breast milk.

You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch, or call 1-800-FDA-1088.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).



THE SENSOREADY PEN: INSTRUCTIONS FOR USE

COSENTYX® (koe-sen'-tix) (secukinumab) Injection 150 mg/1 mL SENSOREADY PEN

Be sure that you read, understand, and follow this Instructions for Use before injecting COSENTYX. Your healthcare provider should show you how to prepare and inject COSENTYX properly using the Sensoready Pen before you use it for the first time. Talk to your healthcare provider if you have any questions.

IMPORTANT:

- **Do not use** the COSENTYX Sensoready Pen if either the seal on the outer carton or the seal on the pen is broken. Keep the COSENTYX Sensoready Pen in the sealed outer carton until you are ready to use it.
- **Do not shake** the COSENTYX Sensoready Pen.
- The caps of the Sensoready Pens contain latex. **Do not handle the Sensoready Pens if you are sensitive to latex.**
- If you drop your COSENTYX Sensoready Pen, **do not use** it if the Sensoready Pen looks damaged, or if you dropped it with the cap removed.
- Throw away (dispose of) the used COSENTYX Sensoready Pen right away after use.
- **Do not re-use a COSENTYX Sensoready Pen.** See “How should I dispose of used COSENTYX Sensoready Pens?” at the end of this Instructions for Use.

HOW SHOULD I STORE THE COSENTYX 150-mg/1 mL SENSOREADY PEN?

- Store your carton of COSENTYX Sensoready Pens in a refrigerator, between 36°F to 46°F (2°C to 8°C).
- Keep the COSENTYX Sensoready Pen in the original carton until you're ready to use to protect from light.
- The COSENTYX Sensoready pens may be stored at room temperature, up to 86°F (30°C), for up to 4 days.
- Write the date the COSENTYX Sensoready pens were removed from and returned to the refrigerator in the space provided on the carton.
- Throw away the COSENTYX Sensoready pen if it has been kept outside of the refrigerator over 4 days.
- COSENTYX Sensoready pen may be returned to the refrigerator only 1 time and must be stored between 36°F to 46°F (2°C to 8°C) until you use it or until it expires.
- **Do not** freeze COSENTYX Sensoready Pens.

Throw away (dispose of) any expired or unused COSENTYX Sensoready pens.

Keep the COSENTYX Sensoready Pen and all medicines out of the reach of children.

Please see additional Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

Visit COSENTYX.com for more information.

COSENTYX Sensoready Pen Parts
(see Figure A):

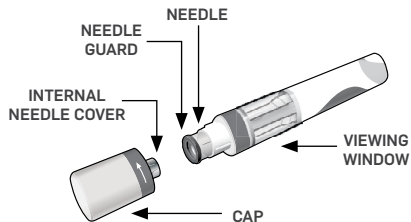


Figure A

The COSENTYX Sensoready Pen is shown above with the cap removed. **Do not** remove the cap until you are ready to inject.

WHAT YOU NEED FOR YOUR INJECTION:

Included in the carton:

- A new COSENTYX® Sensoready Pen (see Figure B). **Each** COSENTYX Sensoready Pen contains **150 mg** of COSENTYX. Check to make sure that you have the correct medicine and dose.
- If your **prescribed dose** of COSENTYX is **150 mg**, you must give **1 injection**.
- If your **prescribed dose** of COSENTYX is **300 mg**, you must give **2 injections**.

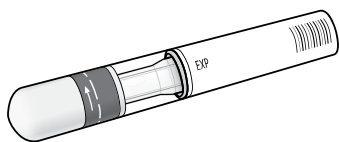


Figure B

Not included in the carton (see Figure C):

- 1 Alcohol wipe
- 1 Cotton ball or gauze
- Sharps disposal container



Figure C

See “**How should I dispose of used COSENTYX Sensoready Pens? (STEP 9)**” at the end of this Instructions for Use.

BEFORE YOUR INJECTION:

Take the carton containing the COSENTYX Sensoready Pen out of the refrigerator and leave it **unopened** for about **15 to 30 minutes before injecting** to allow it to reach room temperature.

STEP 1. Important safety checks before you inject (see Figure D):

- Look through the viewing window. The liquid should be clear. Its color may vary from colorless to slightly yellow.

Do not use if the liquid contains visible particles, is cloudy or is discolored. You may see a small air bubble, which is normal.

- Look at the **expiration date (EXP)** on your Sensoready Pen. **Do not use** your COSENTYX Sensoready Pen if the expiration date has passed.

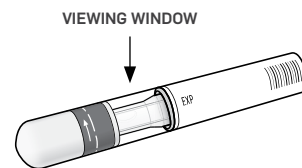


Figure D

Contact your healthcare provider or pharmacist if the COSENTYX Sensoready Pen fails any of these checks.

Please see Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

STEP 2. Choose your injection site:

- The recommended site is the front of the thighs. You may also use the lower abdomen, but **not** the area 2 inches around your navel (belly button) (see Figure E).
- Choose a different site each time you give yourself an injection.
- **Do not** inject into areas where the skin is tender, bruised, red, scaly or hard, or in an area of skin that is affected by psoriasis. Avoid areas with scars or stretch marks.

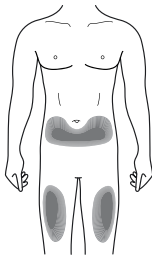


Figure E

- If a **caregiver** or **healthcare provider** is giving you your injection, they may also inject into your outer upper arm (see Figure F).

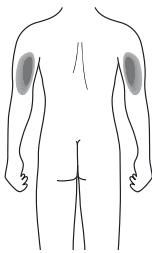


Figure F

STEP 3. Cleaning your injection site:

- Wash your hands well with soap and water.
- Using a circular motion, clean the injection site with the alcohol wipe. Leave it to dry before injecting (see Figure G).
- **Do not** touch the cleaned area again before injecting.

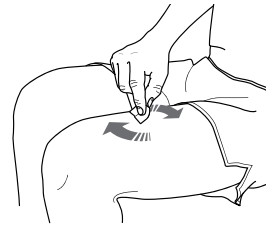


Figure G

YOUR INJECTION:**STEP 4. Removing the cap:**

- Only remove the cap when you are ready to use the COSENTYX® Sensoready Pen.
- Twist off the cap in the direction of the arrow (see Figure H).
- Throw away the cap. **Do not try to re-attach the cap.**
- Use the COSENTYX Sensoready Pen within 5 minutes of removing the cap.

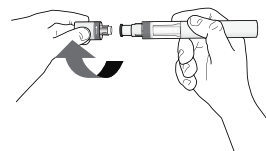


Figure H

Please see Important Safety Information throughout brochure. [Click here for full Prescribing Information, including Medication Guide.](#)

STEP 5. Holding your COSENTYX® Sensoready Pen:

- Hold the COSENTYX Sensoready Pen at 90 degrees to the cleaned injection site (see Figure I).

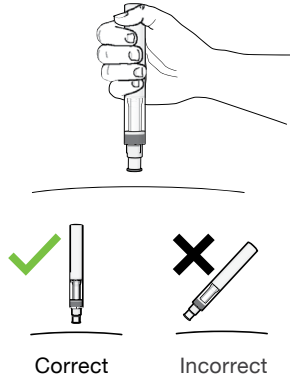


Figure I

IMPORTANT: During the injection you will hear 2 loud clicks:

- The **1st click** indicates that the **injection has started**.
- Several seconds later, a **2nd click** will indicate that the **injection is almost finished**.

You must keep holding the COSENTYX Sensoready Pen firmly against your skin until you see a **green indicator** fill the window and stop moving.

STEP 6. Starting your injection:

- Press the COSENTYX Sensoready Pen firmly against the skin to start the injection (see Figure J).
- The **1st click** indicates the injection has started.
- **Keep holding** the COSENTYX Sensoready Pen firmly against your skin.
- The **green indicator** shows the progress of the injection.

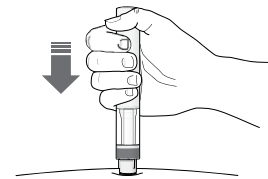


Figure J

STEP 7. Completing your injection:

- Listen for the **2nd click**. This indicates the injection is **almost** complete.
- Check that the **green indicator** fills the window and has stopped moving (see Figure K).
- The COSENTYX Sensoready Pen can now be removed.

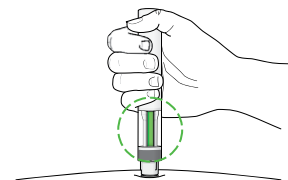


Figure K

Please see Important Safety Information throughout brochure. Click here for full [Prescribing Information](#), including [Medication Guide](#).

AFTER YOUR INJECTION:

STEP 8. Check that the green indicator fills the window (see Figure L):

- This means the medicine has been delivered. Contact your healthcare provider or pharmacist if the green indicator is not visible.
- There may be a small amount of blood at the injection site. You can press a cotton ball or gauze over the injection site and hold it for 10 seconds. Do not rub the injection site. You may cover the injection site with a small adhesive bandage, if needed.

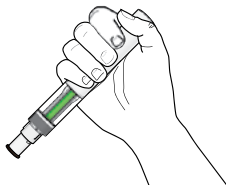


Figure L

If your prescribed dose of **COSENTYX**[®] is 300 mg, repeat steps 1 through 8 with a new **COSENTYX 150 mg/2mL Sensoready Pen**.

How should I dispose of used **COSENTYX Sensoready Pens?**

STEP 9. Put your used **COSENTYX Sensoready Pens** in an FDA-cleared sharps disposal container right away after use (see Figure M). **Do not throw away (dispose of)** Sensoready Pens in your household trash.

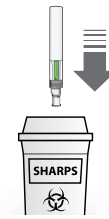


Figure M

If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:

- made of a heavy-duty plastic,
- can be closed with a tight-fitting, puncture-resistant lid without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labeled to warn of hazardous waste inside the container.

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles, syringes, and **COSENTYX Sensoready Pens**.

For more information about safe sharps disposal, and for specific information about sharps disposal in the state that you live in, go to the FDA's website at: <http://www.fda.gov/safesharpsdisposal>.



“The pen is easy for me. I just click and go.”

MARK

ACTUAL PATIENT WITH ANKYLOSING SPONDYLITIS USING THE SENSOREADY® PEN



Cosentyx[®]
(secukinumab)



Visit [COSENTYX.com](https://www.cosentyx.com)



Follow us on Instagram at:
[@COSENTYX \(secukinumab\)](https://www.instagram.com/cosentyx)



Like us on Facebook at [@COSENTYX \(secukinumab\)](https://www.facebook.com/cosentyx)



Find us or follow us on TikTok
[@COSENTYX \(secukinumab\)](https://www.tiktok.com/@cosentyx)

Click here for full [Prescribing Information](#), including [Medication Guide](#).



NOVARTIS

Novartis Pharmaceuticals Corporation
East Hanover, New Jersey 07936-1080

© 2024 Novartis

9/24

FA-11241471